



Paty Face Mask Cover Care & Use Instructions

Thank you for your purchase of our Paty Face Mask Cover, please see below for intended use and care instructions. **Please note that our Paty Face Mask Cover is not intended to be a replacement for a properly fitted N95 respirator. It is intended to be used as a general-use cloth face covering or as a cover to be placed over an appropriate medical grade respirator.** CDC guidelines for the use of Cloth Face Coverings to help slow the spread of COVID-19 can be found here: <https://www.cdc.gov/coronavirus/2019-ncov/prevent-getting-sick/diy-cloth-face-coverings.html>

Recommended Placement:

- Mask should be placed over the nose and mouth, fitting snugly, but comfortably, against the side of the face.
- Ensure the elastic portion is at the top and the pocket edge is on the inside.



Care Instructions:

- Remove and discard filter after one use. We do not currently offer replacement filters. If you don't have access to a non-woven polypropylene material, household items like a coffee filter or air filter, will work as a replacement filter. Please note, even without a filter, your Paty Mask is double layered, and meets the CDC requirements for Cloth Face Covering Guidelines.
- Wash your mask as often as you feel necessary, or once it becomes soiled. Please remove and discard filter prior to washing. Hand washing with soap and warm water is the best way to clean your mask.
- Lay mask out to dry.

Paty, Inc.
4540 South Pinemont Dr., Suite 110
Houston, TX 77041
www.patyinc.com
713-688-7686